

## iScreen™ One Step Drug Card


*Corporate Drug Testing Made Simple.*

- Simple Dip and Read Procedure
- Room Temperature Storage
- Laboratory Accurate
- 1-12 Drug Configurations Available
- Results in Minutes

Instant Technologies, Inc.



*Innovative Drug Screening Solutions*  
**Quality • Choices • Support**

an  inverness medical company



# iScreen™ One Step Drug Card

Corporate Drug Testing Made Simple.


Detect the presence of drugs in minutes.  
It's drug testing efficiency you can count on.

Drug Name:	Abbreviation:	Cutoff Levels:
Amphetamine.....	AMP.....	1000ng/ml
Barbiturates.....	BAR.....	300ng/ml
Benzodiazepines.....	BZO.....	300ng/ml
Cocaine.....	COC.....	300ng/ml
Ecstasy.....	MDMA.....	500ng/ml
Marijuana.....	THC.....	50ng/ml
Methadone.....	MTD.....	300ng/ml
Methamphetamine.....	mAMP.....	1000ng/ml
Morphine.....	MOP.....	300ng/ml
Opiates.....	OPI.....	2000ng/ml
Oxycodone.....	OXY.....	100ng/ml
Phencyclidine.....	PCP.....	25ng/ml
Propoxyphene.....	PPX.....	300ng/ml
Tricyclic Antidepressants.....	TCA.....	1000ng/ml




iScreen Dip Card single and 5 panel shown above.

## Instructions

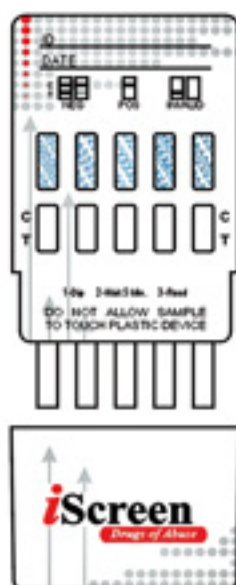

1. Remove iScreen™ test from foil pouch at room temperature. Take off the cap revealing the testing strips.
2. Immerse test vertically in the urine sample for at least 10 - 15 seconds. Do not allow the specimen to touch the device housing.
3. Read at five (5) minutes.  
Results stable up to four (4) hours.

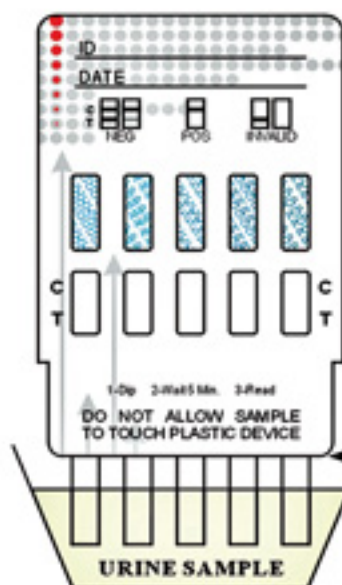

This illustration shows  
a 5-Drug Test Card

Do not allow specimen to  
touch the device housing

### Interpretation of Results:



**Negative**

(2 Lines - Any indication of a line in both test and control fields is to be interpreted as a NEGATIVE)



**Non-Negative**

(1 Line)



**Invalid**

(No line(s) or no control line(s))