



# Procedure Card

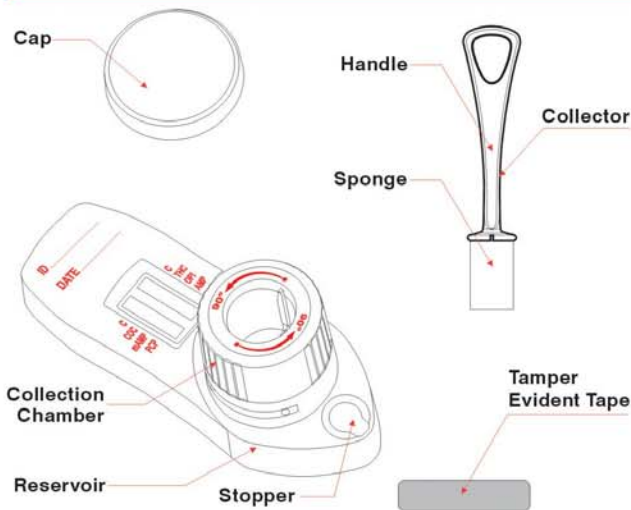


1. Instruct the donor to not place anything in the mouth including food, drink, gum or tobacco products for at least 10 minutes prior to collection.
2. Insert sponge end of collector into mouth. Actively swab the inside of the mouth and tongue with sponge. As soon as the sponge softens slightly, gently press the sponge between the tongue and teeth to ensure complete saturation. The sponge is saturated when no hard spots can be detected. Collect for a total of (3) minutes before removing sponge.




3. Insert collector into device so that the notches on the collector are aligned with the tracks on the inside of the collection chamber. Push collector down into chamber. Turn collector clockwise to lock into chamber. Wait 1 minute.

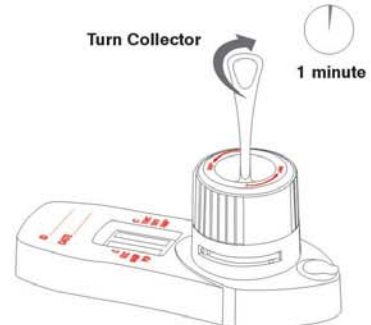

4. Rotate collection chamber 90° counterclockwise.

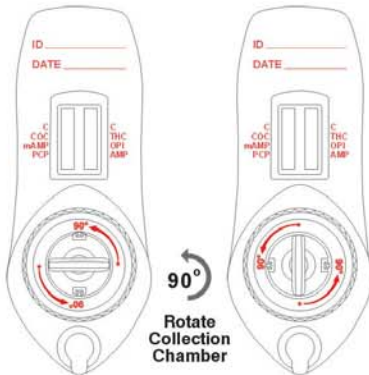


5. Wait 9 minutes and read results.

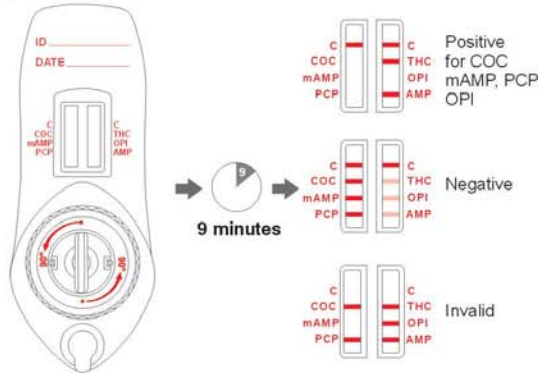

\* All positive results are presumptive and should be confirmed by an alternate method (e.g. GC/MS or GC/MS/MS).

6. If results are positive, remove collector from device and secure reservoir with tamper evident tape. Secure cap onto collection chamber. Send device to laboratory for confirmation. Laboratory can access reservoir through stopper.

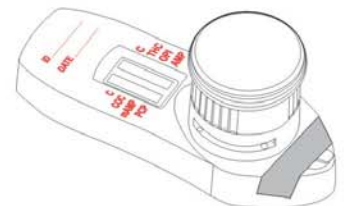

No.: 1155164501

PANTONE 185 CVC

**Attention:** By approving the enclosed design draft, you (ABON's Customer) accept all responsibility for the accuracy of the design. If an error is detected following the printing or manufacturing of a material, you (ABON's Customer) are responsible for the cost of any inventory which is deemed unsuitable for sale.

☒ US

☐ OUS

☐ OTHER

Description	SD-006 procedure card	Part Number	1155164501	Size	270X200mm
Printing Contents		L Number		Size	
Designer	Delia	Design Date/Version	June 19, 2006 / G	Mold Number	
Artwork checked by		Material	350g 白卡	Checked by	
Approved by Customer		Approved by Marketing/Sales			
Approved by P.M.T.		Approved by QA		Effective Date	