

Q.E.D.[®]

SALIVA ALCOHOL TEST

The On-The-Spot Leader In Alcohol Testing . . .



- Simple
- Accurate
- Reliable
- D.O.T. Approved
- CLIA-Waived
- *In Vitro* Diagnostic Use



The disposable Q.E.D.[®] Saliva Alcohol Test is a revolutionary breakthrough in quantitative on-site alcohol detection.

Distributed by:

Q.E.D.[®]

SALIVA ALCOHOL TEST

1-800-869-3538

Three easy steps:



1. Swab mouth to collect saliva.



2. Insert collector into test.



3. Read color bar.




OraSure Technologies, Inc.

diagnostic solutions for the new millennium

220 East First Street • Bethlehem, PA 18015
www.orasure.com

On-Site Simplicity with Clinical Lab Accuracy . . .

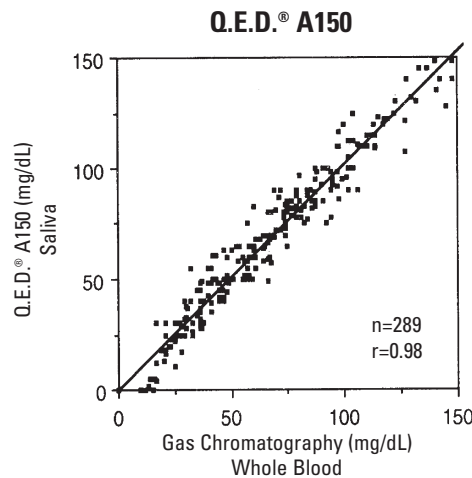

Data on file. ¹Study conducted by Thomas Jefferson University Hospital, Department of Emergency Medicine.
²Study conducted by Rutgers University, Center of Alcohol Studies.

The Q.E.D.[®] test can be used as an accurate screen in place of blood tests. In clinical trials, saliva alcohol levels measured with the Q.E.D.[®] tests demonstrated high correlation to blood analyzed by gas chromatography ($r=0.98$)¹ and produced no false positive or false negative results².

It's Reliable, Fast, Simple . . .

- Built-in QA Spot[™] assures quality control for every test
- Quantitative results in minutes
- Long shelf life
- Reads like a thermometer
- No instrumentation
- Non-invasive

The Q.E.D.[®] test is used for . . .

- Pre-employment or pre-duty
- Random testing
- Post-accident
- Reasonable cause
- Follow-up testing

**Worksite testing
for DOT or drug-free
workplace programs**

- Testing upon admission
- Interventions
- Counseling sessions
- Random testing

**Alcohol treatment
centers**

- Pre-discharge
- Admission screening
- Routine trauma treatment protocol
- Seizure or unconscious victims

**Emergency/trauma
departments**

Additional Information . . .

- Available testing range:
→A150 (0-145 mg/dL, 0-0.145%)
- DOT Screening Test Technician Certification available from Nationwide Network of Trainers or **NEW** DOT-approved training video
- Toll Free Technical Support—Monday thru Friday 8 a.m. - 6 p.m. (eastern time)

OraSure Technologies

Product Number

31150(B,C)
31050S
31150T
31150X
31000V

Description

Q.E.D.[®] A150—case of 30 tests(B), case of 100 tests(C)
Q.E.D.[®] DOT Ethanol Control (also 31150S, 31350S)
Screening Test Technician (STT) Training Video Kit
Additional STT Training Student Workbooks
Q.E.D.[®] Procedure Video—case of 10